

Instructions to use MEDIPHOT X-PIR/RP Laserprint Film in Imation 969 printers

Before inserting the MEDIPHOT DLP-M2 tray the black rubber that supports the teeth of the tray should be replaced:


Position of the rubber support

At the most left side of the input drawer, right before the roll that will wind the coversheet you will find a black solid rubber, partly with teeth of about 1 cm. (See pictures below)


Reason for replacement

The slightly different shape of the MEDIPHOT tray requires an adapted rubber with room for the rib in front of the tray.
(This rib is caused by the magnetic stripes)

Differences between MEDIPHOT and IMATION tray: Left = MEDIPHOT with rib, Right = Imation/EK:

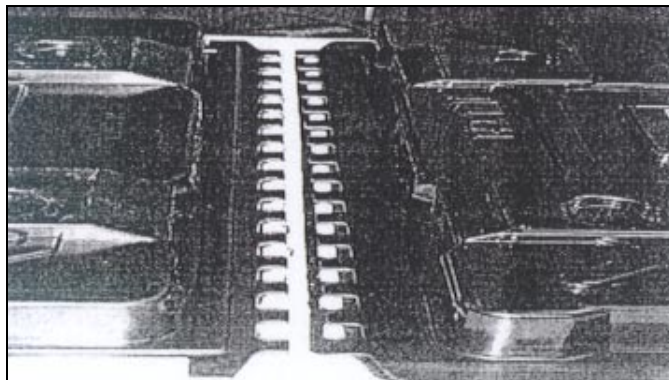

Different rubber types: Left = Imation/EK, Right MEDIPHOT with space for the rib.

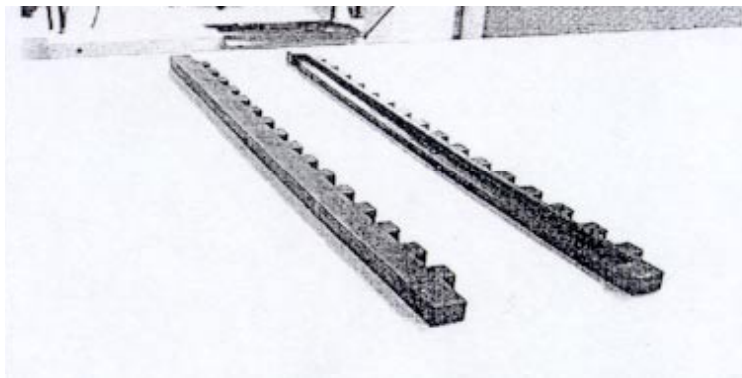

The new rubber is compatible with the IMATION trays. If, after a demo, the customer wants to continue using his old stock of IMATION-film, there is no need to re-place the original rubber.